
FICHA TÉCNICA

MPR076

MASCARILLA AUTOFILTRANTE
FILTERING HALF MASK

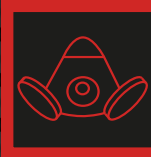FICHA TÉCNICA
DATASHEET

1720 FFP2

NR

MADE IN SPAIN

WWW.PRODUCTOSCLIMAX.COM

1720 FFP2 NR
MASCARILLA AUTOFILTRANTE
FILTERING HALF MASK

Descripción
Description

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Climax filtering half masks have been designed to effectively retain dust particles in suspension, mist and smoke. They are very light, comfortable and offer a very low resistance to breathing. According to their efficacy, they are classified in three levels of protection: FFP1, FFP2 and FFP3.

Type of uses for the half mask **1720 FFP2 NR**: Sanding of old coats of paint, coatings, scraping off, adhesive sanding, grinding, cutting and drilling of wood, smoke particles from soldering, waste management, pesticide fumigation and use/cleaning of animal feeding systems.

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When from the risk assessment it is derived that in the development of the activity care procedures are carried out in which **bioaerosols** can be generated in **high concentrations**, the use of **filtering half masks against particles** is recommended.

The respiratory protection generally recommended for personnel who may be in contact within 2 meters with cases under investigation or confirmed cases is an **FFP2 / FFP3** filtering half mask.

Self-filtering masks **should not be reused** and should therefore be discarded after use. The user **can not** apply their own **disinfection** methods since the effectiveness of the equipment may be affected.

In order to avoid infection, people who show **symptoms** or have been **diagnosed** with the disease must wear a half-mask.

In the event that they wear a filtering half-mask, **in no case will this include an exhalation valve** since in this case the air is exhaled directly into the environment without any type of retention and the spread of the virus would be favored.

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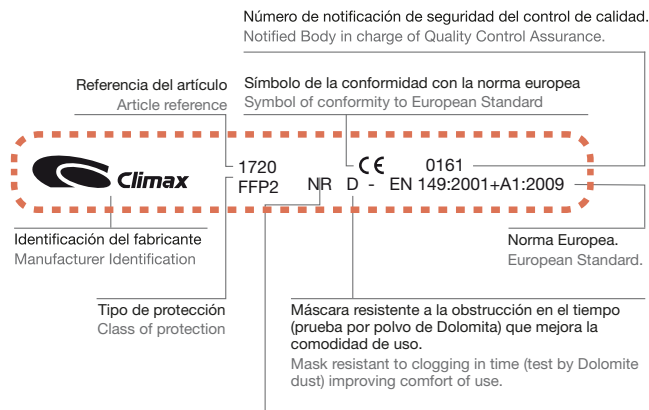
This equipment should not be used in the following situations:

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- Oxygen-Deficient atmospheres. (less than 19,5%)
- Immediately dangerous to life or health (IDLH)
- Unknown contaminants.
- Gases and vapours.

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Skilled persons who have been adequately trained or who use the belt under the supervision of a competent supervisor may only use the filtering half masks.

05/2020